




CADD PRIZM VIP PUMP INSTRUCTIONS

ALL MODES

TO CHANGE BATTERIES DURING AN INFUSION


If “**9-volt Battery Low**” or “**9-volt Battery Depleted**” appears in the display, or if “**Battery Low**” appears on the main screen, you should change the battery.

1. Stop the pump by pressing the **START/STOP** key.

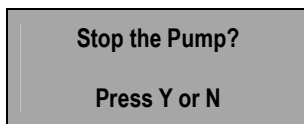

Press “**Y**”

2. Press the white button on the battery door and slide the door forward. Remove used battery and discard appropriately.
3. Match the + and – markings on the new battery with the markings on the pump. Insert the battery. The pump will beep if the battery is inserted correctly.
4. Replace the battery door. Pump will power up automatically and go through self test (approx. 50 sec.).
5. Press the START/STOP key

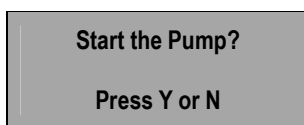

Press “**Y**”

*****Green light at top of pump will begin flashing and the “RUNNING” message will appear in the screen. Screen will go blank after approx. one minute.**