



CADD PRIZM VIP PUMP INSTRUCTIONS

CONTINUOUS MODE

TO CHANGE A MEDICATION BAG

1. Press **STOP/START**, followed by **“Y”**.
2. Remove cassette, and replace with new cassette.
3. Follow the displays below:

PUMP DISPLAY

KEY PRESS

Admin set latched.
Upstream sensor on.

NEXT to continue

Press **NEXT**

Reset Reservoir
Volume to xxxml?

Press Y or N

If volume correct,
Press **“Y”**

Prime Tubing?

Press Y or N

Press **“Y”** or **“N”**
depending on whether
the tubing is already pre-
primed.

******If priming, Press and hold the “Y” key to prime, release when fluid reaches the tip of the tubing.*****

Continue Priming?

Press Y or N

If done priming,
Press **“N”**

Start the Pump?

Press Y or N

Press **“Y”**

Pump will start up and scroll through the program once again. You may speed this up by pressing the NEXT key through each parameter until the following screen:

***CON 6220C ***
9V Battery RUNNING
Res Vol xxx.x ml
Press NEXT to advance

****green light will be blinking at the top of the pump**

*******Pump screen will go blank after a minute to conserve battery. To bring screen back on, Press NEXT.**