



CADD PRIZM VIP PUMP INSTRUCTIONS

CONTINUOUS MODE

TO ADD A NEW MEDICATION BAG

1. Attach tubing cassette to pump.
2. Remove battery door, insert 9Volt battery in correct position (pump will beep when battery is inserted correctly). Replace battery door.
3. Pump will power up and automatically review program.
4. Pump will beep six times when power up is complete.
5. Follow the displays below:
(****Pump screen goes blank to preserve battery – press NEXT key to bring display back)

PUMP DISPLAY

***CON	6220C ***
9V Battery	STOPPED
Press NEXT to advance	

Start the Pump?
Press Y or N

KEY PRESS

If you wish to check program again – use the NEXT key to scroll through.
When patient is connected,
Press STOP/START.

Press “Y”

Pump will start up and scroll through the program once again. You may speed this up by pressing the NEXT key through each parameter until the following screen:

***CON	6220C ***
9V Battery	RUNNING
Res Vol	xxx.x ml
Press NEXT to advance	

****green light will be blinking at the top of the pump**