



CADD PRIZM VIP PUMP INSTRUCTIONS

PCA MODE

TO CHANGE PAIN MANAGEMENT BAG

****NOTE You will need the key to do a bag change.**

Stop the infusion by pressing STOP/START.

Stop the Pump?
Press Y or N

Press "Y"

***PCA 6210G ***
9V Battery STOPPED
Press NEXT to advance

Unlock and remove the tubing cassette. Replace with new tubing cassette and lock.

Cassette Locked
NEXT to continue

Press "NEXT"

Reset Reservoir
Volume to xxx.x ml?
Press Y or N

If volume correct,
Press "Y"

Prime Tubing?
Press Y or N

Press "N"
(pain management is already primed)

Start the Pump?
Press Y or N

If not clearing boluses, Press "Y"
If you wish to clear boluses,
Press "N" and skip to "CLEARING BOLUSES" on page 2.

Pump will review the program once more and start up.

***PCA 6210G ***
9V Battery RUNNING
Res Vol x x.x ml
Press NEXT to advance

Green light at top of pump will be blinking and screen will go blank after a minute.

CLEARING BOLUSES

***PCA 6210G ***
9V Battery STOPPED
Press NEXT to advance

Press “NEXT” until the following screen appears:

Dose Counters
Given/Attempt: x/x
since xx/xx/xx xx:xx
Press ENTER to clear

Press “ENTER”

Milligrams Given
xx.x mg
since xx/xx/xx xx:xx
Press ENTER to clear

Press “ENTER”

***PCA 6210G ***
9V Battery STOPPED
Press NEXT to advance

Press “STOP/START”

Start the Pump?

Press Y or N

Press “Y”

Pump will review the program once more and start up.

***PCA 6210G ***
9V Battery RUNNING
Res Vol x x.x ml
Press NEXT to advance

Green light at top of pump will be blinking and screen will go blank after a minute.