



CADD PRIZM VIP PUMP INSTRUCTIONS

PCA MODE

TO START A NEW PAIN MANAGEMENT BAG

1. Attach tubing cassette to pump. ****NOTE** You will need to lock the cassette bar in place with the key or the pump will not start.
2. Remove battery door, insert 9Volt battery in correct position (pump will beep when battery is inserted correctly). Replace battery door.
3. Pump will power up and automatically review program. (You will be able to review the pump settings again after the self test is complete).
4. Pump will beep six times when power up is complete.
5. Follow the displays below:
(****Pump screen goes blank to conserve battery – press NEXT key to bring display back)

PUMP DISPLAY

KEY PRESS

***PCA 6210G ***
9V Battery STOPPED
Press NEXT to advance

Press “START”

Start the Pump?
Press Y or N

Press “Y”

**** Pump will now review the settings again and automatically lock to LL1. The following screen will appear, but then will go blank. Press NEXT at any time to bring back the display – and NEXT consecutively to check the parameters while pump is running.**

***PCA 6210G ***
9V Battery RUNNING
Res Vol xxx.x ml
Press NEXT to advance

***green light will be blinking at the top of the pump while running and screen will go blank.

PLEASE NOTE: Pump will begin to beep at 1ml increments beginning at 5mls remaining to notify of “Low Reservoir Volume”. Both the yellow and green lights at the top of the pump will blink during this time. When infusion is finished and reservoir volume is zero, pump will begin with repeated Double Beeps. The screen will appear:

Reservoir Volume
is Zero
Press NEXT to Continue