



CADD PRIZM VIP PUMP INSTRUCTIONS

PCA MODE

SITE CHANGE WITHOUT CHANGING BAG

*****Prepare your equipment for site change before stopping the pump to minimize time off of the infusion.**

Stop the infusion by pressing STOP/START.

Stop the Pump?
Press Y or N

Press “Y”

***PCA 6210G ***
9V Battery STOPPED
Press NEXT to advance

*******While pump is stopped, disconnect tubing to med bag (keeping the tip uncontaminated) and proceed with site change. (Pump may beep during the time that it is stopped – this is just a reminder that pump is stopped.) Reconnect tubing to new site and restart the pump: If screen has gone blank, just press NEXT to bring it back, then press STOP/START.**

Start the Pump?
Press Y or N

**If not clearing boluses, Press “Y”
If you wish to clear boluses,
Press “N” and skip to “CLEARING
BOLUSES” on page 2.**

Pump will review the program once more and start up.

***PCA 6210G ***
9V Battery RUNNING
Res Vol x x.x ml
Press NEXT to advance

Green light at top of pump will be blinking and screen will go blank after a minute.

CLEARING BOLUSES

***PCA 6210G ***
9V Battery STOPPED
Press NEXT to advance

Press “NEXT” until the following screen appears:

Dose Counters
Given/Attempt: x/x
since xx/xx/xx xx:xx
Press ENTER to clear

Press “ENTER”

Milligrams Given
xx.x mg
since xx/xx/xx xx:xx
Press ENTER to clear

Press “ENTER”

***PCA 6210G ***
9V Battery STOPPED
Press NEXT to advance

Press “STOP/START”

Start the Pump?

Press Y or N

Press “Y”

Pump will review the program once more and start up.

***PCA 6210G ***
9V Battery RUNNING
Res Vol x x.x ml
Press NEXT to advance

Green light at top of pump will be blinking and screen will go blank after a minute.