



CADD PRIZM VIP PUMP INSTRUCTIONS

TPN MODE

TO CHANGE TPN BAG

1. Press STOP/START, the “Y” to stop the pump.
2. Remove tubing cassette from pump.
3. (If changing battery, do so now and let pump completely power up before continuing).
4. Attach new tubing cassette to pump.
5. Follow the displays below: (****Pump screen goes blank to conserve battery – press NEXT key to bring display back)

PUMP DISPLAY

ACTION

Admin set latched.
Upstream sensor On.

NEXT to continue

Press “NEXT”

Reset Reservoir
Volume to xxx.x ml?

Press Y or N

Press “Y”

Prime Tubing?

Press Y or N

Press “Y”, then press and hold the “Y” key until fluid reaches the end of the tubing and then release to stop.

Continue Priming?

Press Y or N

If done priming, press “N”.

Start the Pump?

Press Y or N

Press “Y”

***Pump will go through the parameters again and start up. The screen will appear as follows:

***TPN 6230C ***
9V Battery RUNNING
Res Vol xxxx.x ml
Press NEXT to advance

*** Green light will be blinking at the top of the pump and the screen will go blank.